www.nevercold.at | www.nevercold.de

NECO

WARMING PADS

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**DE
EN**

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Geruchlos
Odorless

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NECO MANUAL

The **NECO** warming pads are the punctual heat therapy for everyday usage. It is always at hand and ready to use.

NECO offers independent beneficial heat treatment for muscle relaxation, advanced metabolism and pain reduction.

Doctors and therapists use the **NECO** complementary to their treatment to achieve better results.

Instant Heat

Air activated

Ready to use

Odorless

Disposable

1

1 Piece

Please read all instructions and warnings carefully before use.

INSTRUCTIONS

- Tear open outer wrapper only when ready to use. **NECO** develops its therapeutical warmth in between 15 to 30 Minutes.
- If the product feels uncomfortably hot, stop use or wear it over clothing.
- **NECO** will stay warm for up to 16 hours. However it can be shortend depending on storage conditions or when using external exposure.

ATTENTION:

THIS PRODUCT CAN CAUSE BURNS.

CHECK SKIN FREQUENTLY DURING USE.

DO NOT USE

- While sleeping.
- With pain rubs, medicated lotions, creams or ointments.
- On the same place for an extended period of time.
- In tight fitting apparel that presses NECO against the skin.
- With other heating devices or during a MRI (magnetic resonance imaging). Do not heat it in a microwave.
- If NECO contents leak and/or the wrapper is damaged or torn.

WARNINGS

Not recommended for infants and children as people who cannot feel heat or have a sensitive skin, diabetes, scars, open wounds, circulatory or vascular disorders. Extra precaution for children, elderly or physically challenged people. If in doubt, consult a doctor.

FOR EXTERNAL USE ONLY.

Do not put the warming pad in your mouth. In case of ingestion, rinse mouth with water and consult a doctor. If contents come into contact with eyes or on skin, do not scratch. Wash with water and consult a doctor.

STORAGE:

Store in a cool place away from direct sunlight. Keep out of reach of children and pets.

CONTENTS:

Iron powder, water, activated charcoal, salt, vermiculite and polymer.

DISPOSAL

Discard with regular household trash.

HIGHEST TEMPERATURE:

158°F (70°C). Average Temperature: 140°F (60°C).

THE NECO STRAPS MAXI BLACK/WHITE


The **NECO STRAPS** are a consistent opportunity to use the **NECO** warming pads punctual in stationary or mobile conditions. They consist of skin-friendly cotton-strech and offer with their velcro fastener system a wide area of application to position the **NECO** warming pad at the body. This way the therapeutical heateffect is easy to use and regulate in everyday conditions.

USAGE OF THE NECO STRAP IN COMBINATION WITH THE NECO WARMING PADS:


- Tear open outer wrapper when ready to use. **NECO** develops its therapeutical warmth in between 15 to 30 Minutes.
- Put the **NECO** warming pad into the intended bag of the **NECO STRAP**.
- Apply the **NECO STRAP** so that the required part of the body get provided with heat.
- If the product feels uncomfortably hot, stop use or change the position of the pouch with the **NECO STRAP**.

neco STRAP


USAGE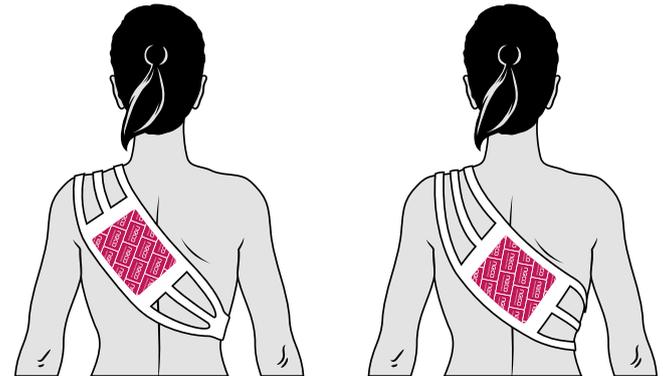

**FOR THE HEAD, NECK AND
SHOULDER PAIN**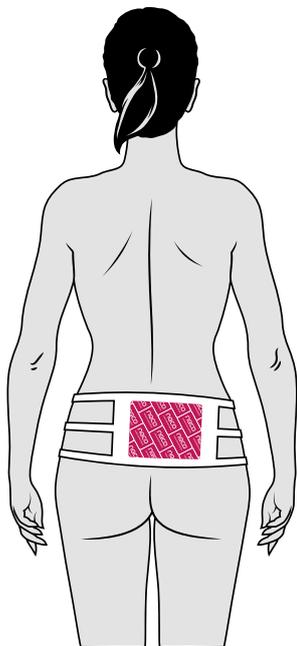

**FOR BACK PAIN
(LWS SYNDROME)**

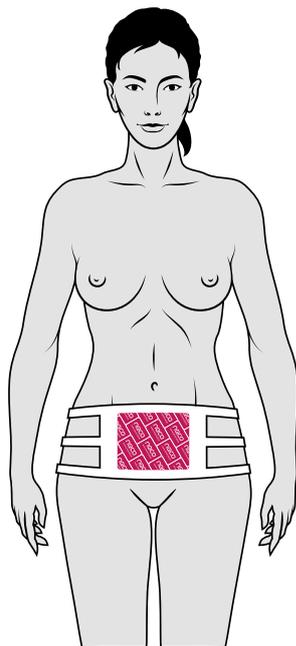**FOR MENSTRUATIONAL
COMPLAINTS**

BENEFITS

When the **NECO** warming pad is used with the **NECO STRAP** it fits tightly to the body so you can use it during sports or an active everyday life.

The extension element offers an additional option for the individual usage of the **NECO STRAP**.

The **NECO STRAP** is, like the extension element, reusable and easy to wash in the washing machine by wool- or fine wash cycle.

WARNINGS

The warnings of the **NECO** warmingpads also apply when it's used with the **NECO STRAP**!

NECO

WARMING PADS

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Stephan Kraus Produktvertrieb e.U.

info@nevercold.at

www.nevercold.at

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info@nevercold.de

www.nevercold.de

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Gisela Kraus GmbH